

Use of Duodopa[®] following PEG surgery

co-careldopa


Morning routine:

1. Take the Duodopa cassette out of the fridge at least 20 minutes before starting the pump.
2. Attach the cassette to the pump:
 - Fit the hooks of the cassette onto the pivot pins at the pump's base.
 - Place the pump on a level surface
 - Steady the cassette and put a coin in the slot of the lock knob.
 - Lock the cassette by pushing the coin or the Duodopa key and turning it 90 degrees counter-clockwise until it stops.
3. Remove stopper from the pump tube.
4. Remove the **RED** cap from the cassette tubing.
5. Connect the cassette tubing to the PEG tube via the luer lock, gently rotating the green luer connector anti-clockwise.
6. Turn the pump on:
 - Press and hold down the **ON OFF** button for 3 seconds.
 - Press and hold down the **STOP START** button for 3 seconds.
7. Give the Morning Dose:
 - Press the **MORNING DOSE** button. The set morning dose is shown in the display. Press the **MORNING DOSE** button again and the pump will administer the morning dose.
8. Continuous Dose:
 - The pump will automatically deliver the continuous dose after the morning dose has been administered.
9. Extra Dose:
 - Press the **EXTRA DOSE** button once. (This is to be used if you find you need an extra dose. It should only be required up to 4 times a day. If you find that you require this more often, you should contact your Nurse or Solvay Nurse as you may need your continuous dose increased. The extra dose can only be given again once an hour has passed).



Evening routine:

1. Press the **STOP START** button for 3 seconds followed by the **ON OFF** button to turn the pump off.
2. Disconnect the pump from the PEG tube by gently rotating the green luer lock connector clockwise.
3. Slowly flush the PEG tube with 10-30ml drinking water until the tube is clean and clear (A). Alternatively, use a 3-way connector to flush (B).
4. Detach the cassette from the pump:
 - Place the pump with the cassette upright on a level hard surface.
 - Put a coin or the Duodopa key in the slot of the lock knob and turn 90 degrees clockwise. The lock knob will pop out slightly when the cassette is unlocked.
 - Remove the hooks of the cassette from the pivot pins of the pump.
 - Discard the used cassette if it is not to be used later the same day.

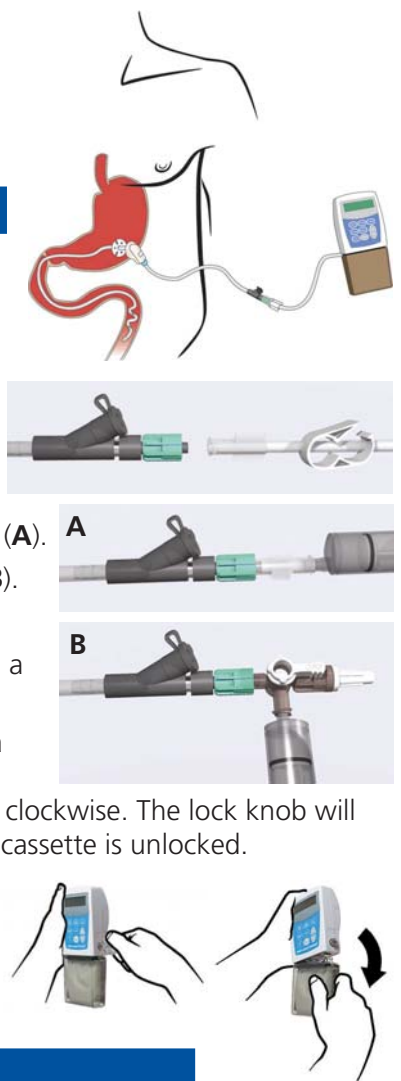

Weekly maintenance:

Once a week flush the outer tube of the PEG with 10-30ml drinking water using the purple connector supplied.



© Registered trade mark
Date of preparation: October 2006

If you have any questions or queries, please do not hesitate to contact the 24hour Duodopa Helpline Nurses on:
0800 458 4410